

SOL-CARE™ Safety Hypodermic Needle

Putting Safety First

SOL-CARE™ Safety Hypodermic Needle is a sharp effective needle with the added benefit of an integrated safety device designed to minimise the risk of accidental needlestick injuries.

Thumb Activation



After injection, place your thumb on the base of the safety sheath and push the cover down over the needle until an audible click is heard.

Hard Surface Activation



After injection, engage the Sol-Care Safety Hypodermic Needle by gently pressing the coloured sheath against a suitable hard flat surface until an audible click is heard.



Audible and visual confirmation will indicate that the needle is captured inside the safety cover.



Dispose of the device into an appropriate sharps container.

User Benefits

- ISO colour coded - Protective sheaths are fully colour coded to reflect the needle gauge size allowing for ease of identification
- Bevel up oriented needle - Allows for low angle injections
- Easy positioned safety sheath - The protective sheath has a flexible hinge allowing it to be folded back against the syringe for maximum visualisation of the needle bevel and injection site
- Patent locking design - The safety mechanism is a ball and socket joint that audibly clicks into position when the safety device is engaged
- Standard luer hub - Compatible with both luer lock and luer sip syringes
- Protective safety sheath - Correlates with the length of the needle in order to optimise visualisation of the needle bevel
- Intuitive single handed activation - Allowing for the safety sheath to be engaged either utilising hard surface activation or using the thumb pad

Product Specifications

SOL-CARE™ Safety Hypodermic Needles

Colour Code	Standard Gauge	Needle Length (inches)	Reference No.	NPC Code	Colour Code	Standard Gauge	Needle Length (inches)	Reference No.	NPC Code
Yellow	30	1/2	SN3012	FTR1985	Orange	20	1	SN2010	FTR1965
Grey	27	5/8	SN2758	FTR1984	Yellow	20	1 1/2	SN2015	FTR1966
Grey	27	1/2	SN2712	FTR1981	Orange	20	2	SN2005	FTR1964
Grey	27	3/4	SN2734	FTR1983	Light Orange	19	1	SN1910	FTR1962
Grey	27	1 1/4	SN27125	FTR1982	Light Orange	19	1 1/2	SN1915	FTR1963
Brown	26	3/8	SN2638	FTR1980	Light Orange	19	2	SN1920	FTR1987
Brown	26	1/2	SN2612	FTR1979	Pink	18	1	SN1810	FTR1959
Orange	25	5/8	SN2558	FTR1978	Pink	18	1 1/2	SN1815	FTR1960
Orange	25	1	SN2510	FTR1976	Pink	18	2	SN1820	FTR1961
Orange	25	1 1/2	SN2515	FTR1977					
Blue	23	1	SN2310	FTR1974					
Blue	23	1 1/4	SN23125	FTR1969					
Blue	23	1 1/2	SN2315	FTR1975					
Black	22	1	SN2210	FTR1972					
Black	22	1 1/2	SN2215	FTR1973					
Black	22	2	SN2205	FTR1971					
Green	21	1	SN2110	FTR1968					
Green	21	1 1/2	SN2115	FTR1970					
Green	21	2	SN2105	FTR1967					

